



PHYSION[®]
PROTOCOLS

**INTRAVESICAL
ADMINISTRATION
OF EMDA-MMC
IN BCG FAILURE PATIENTS**

- Mitomycin: dilute 40 mg of MMC in 100 ml of distilled water under a laminar flow hood
- Start the protocol within 2-4 weeks from the endoscopic resection
- Before the 1st instillation: URINE CULTURE

INDUCTION

- 1 instillation per week for 6 weeks (total 6 EMDA MMC treatments)
- After 45 days from the last instillation the following exams have to be performed:
 - URINARY CYTOLOGY on 3 samples
 - VESICAL BIOPTIC MAPPING
 - PAP test should be performed on urine released spontaneously and by bladder wash with physiological
 - URO-CT/ECO



PHYSION®

MAINTENANCE

- 1 instillation per month for 6 months (total 6 EMDA MMC treatments)
- After 45 days from the last instillation the following exams have to be performed:
 - URINARY CYTOLOGY on 3 samples
 - VESICAL BIOPTIC MAPPING
 - PAP test should be performed on urine released spontaneously and by bladder wash with physiological
 - URO-CT/ECO

CHECK EXAMS

- To be repeated every 3 months till the end of the follow-up:
 - URINARY CYTOLOGY on 3 samples
 - URETHROCYSTOSCOPY
- To be repeated every 6 months till the end of the follow-up:
 - URO-CT/ECO



PHYSION®